Thank you for choosing HABYS products and congratulations on your purchase. We recall that the proper use and maintenance of the products ensure full satisfaction of their use for many years.

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I. GENERAL SAFETY AND USE OF THE TABLE

1. Please read the instructions before using the product.
2. Before using, place the product on a flat surface.
3. Before use, make sure that the product does not wobble or tilt, and that all elements are properly attached and tightened.
4. Do not use the product if any of the product components are malfunctioning or damaged.
5. Do not stand or jump on the product.
6. Do not put fingers or other parts of the body into the area of the regulation mechanisms.
7. Do not expose the product to strong sunlight.
8. Do not use the product in extreme temperature and humidity conditions.
9. Do not allow the product to be loaded with a pressure greater than the maximum specified in the technical parameters, ie 200 [kg].
10. After bringing the table from a cold room, connect it to the mains after waiting 15 minutes.
11. It is recommended that the table be fully adjusted in height from minimum to maximum to ensure even distribution of the lubricant inside the motor. This ensures proper table maintenance and extends the engine life. The operation should be repeated 2 or 4 times min. once a week.
12. Do not remove the plug from the socket by pulling the cord.
13. Use the product only in accordance with its intended purpose, in the event of use other than that specified in this manual, there is a serious risk of injury, damage to health and damage to the product.
14. Do not make any modifications or repairs to the product, do not disassemble the product.
15. In the event of finding any defects, immediately contact the manufacturer and do not use the product until the defect is remedied.
16. No contraindications for the use of the product only after meeting the above recommendations.
17. Each serious incident related to the device should be reported to the manufacturer and the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

II. INTENDED USE OF THE PRODUCT

The Habys Upright Chiro Z7 stationary table is a professional table for chiropractic, manual techniques and osteopathy. Thanks to a wide range of adjustments in various planes, segments for "drop" techniques and an electric top lifting system, the table is a perfect complement to the therapist's work.

Due to its design solutions and functions, the product should be used by personnel qualified in the field of manual therapy, chiropractic or osteopathy.

III. CLEANING AND MAINTENANCE


The metal structure of the table should be cleaned with a damp cloth and mild cleaning agents. The elements of the upholstery should be cleaned with generally available agents intended for cleaning upholstery. Do not use harsh chemicals and those based on alcohol.

IV. CONSTRUCTION AND TECHNICAL DATA

Product components:

1. Frame - strong and durable structure made of a steel bearing frame covered with a varnish coating.
2. Support elements - made of 40 [mm] integral foam.
3. Electric system for lifting the table top in the range of 55 - 97 [cm], adjustment by means of:
 - central control frame;
 - handheld remote (additional option).

4. Wheels with an additional PU layer and a locking system.
5. The lumbar section adjustment system based on 2 gas springs enabling:
 - up / down movement (tilting) in the range of -40° to 22° ;
 - lateral movement (swing) in the range -18° to 18° .
6. The headrest adjustment system based on a gas spring that enables:
 - up / down movement (tilting) of the entire frame headrest in the range of -40° to 30° ;
 - additional movement of the headrest itself in the range of 0° to 30° .
7. A system that enables traction for the cervical and lumbar spine in the total range of 13 [cm].
8. A system that enables the "drop" technique to be carried out on the lumbar, thoracic and cervical spine.
9. Armrest adjustment system in the range of 0-20 [cm].

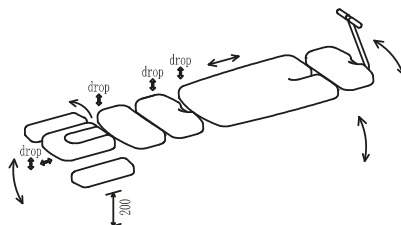Width: 56 [cm]

Length (depending on the user setting): 185 [cm]

Height (depending on user settings): 55 - 97 [cm]

Weight: 130 [kg]

Loading capacity: 200 [kg]

Power supply: voltage 230V, 50 / 60Hz, output 5V 1A

V. INSTALLATION

1. Unpacking.

Unpack the table. Check that it has not suffered any damage during transport. If it is damaged, do not connect it and notify your dealer.

2. Connection to the power network.

Before inserting the plug into a power outlet, make sure that:

- the socket is properly grounded and complies with the applicable regulations;
- the socket is able to withstand the maximum power load of the device, as given in the Technical Data point;
- the voltage of the electric current in the network is consistent with the data given in the Technical Data point;
- the socket corresponds to the plug. If not, replace the socket.

Do not install the devices in the open air. Electric actuators may work not longer than 2 minutes. Restart is possible after 18 minutes of rest. After installation of the table the mains socket should be easily accessible. Do not use the extension cords. The cable should not be bent. The cable of the power supply may be replaced only by an authorised specialist. No independent repairs of the electric actuators should be performed. Avoid direct sunlight, UV radiations, rain, extensive dust, moisture.

Attention!

The manufacturer bears no responsibility for the product if the above-mentioned rules are not followed.

VI. PRODUCT USE



1. Securing the patient.

The table is equipped with 2 straps (included) to secure the patient against further work. The belts should be attached using special holders on both sides of the table.



2. Adjusting the table height with the steering frame.

The steering is done by moving and holding any point of the frame with the foot vertically (up or down). The steering frame is located around the base of the table. By sliding the frame down, the table top rises. By shifting the frame upwards, the table top is lowered.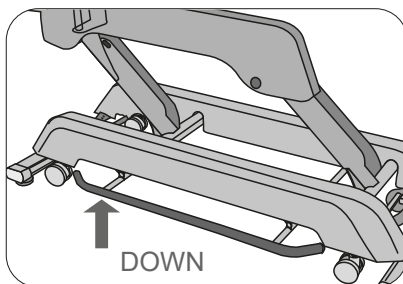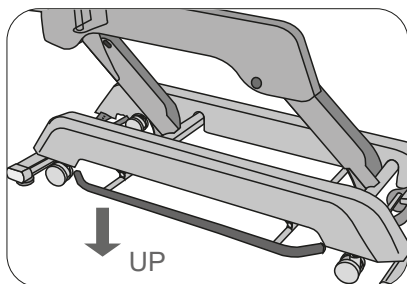

3. Adjusting the table height with a manual remote control (available as an option).

The control is done by pressing one of the buttons marked with arrows - up / down.

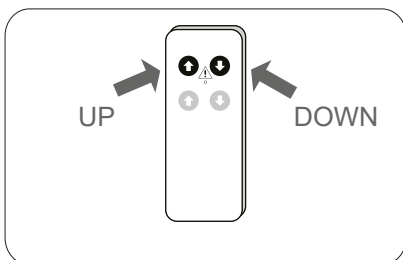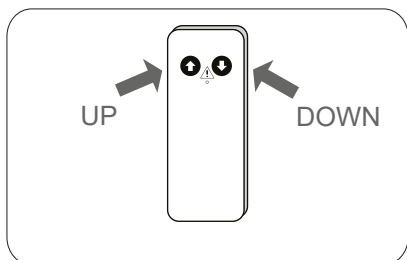4. Headrest adjustment.

The headrest adjustment system is based on the operation of a gas spring, after releasing the yellow lever, set the headrest in the desired position.


Fig. 1 | Fig. 2

For horizontal movement (traction), unlock the knob on the right side of the headrest frame. After finishing work in this segment, lock the knob according to the diagram.

Fig. 3 | Fig. 4

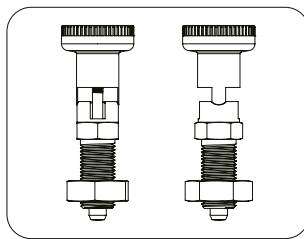5. Adjusting the armrests.

Adjustment is made by releasing the lock and moving the armrest to the desired height.

Fig. 5 |

6. Using the drop function.

Before starting the drop techniques, lay the patient on his stomach, make sure that the patient's position is safe, all belaying elements (belts, headrest, armrests) have been adjusted to the patient's anatomy. The technique should be performed only by qualified personnel, improper execution of the technique may result in permanent injury to the patient.

Drop for the cervical section (Fig. 6).

Raise the cervical section using the lever located at the front of the headrest.

Perform the cervical vertebrae drop technique by rapidly and vigorously compressing the cervical segment, which will force the cervical section of the table to drop.

Drop for the thoracic section (Fig. 7).

Raise the chest section by pressing the lever downwards (for the therapist's convenience, the lever is located on both sides under this table section).

Perform the thoracic vertebrae drop technique by pressing the thoracic segment quickly and vigorously, which will force the table section to fall down.

Drop for the lumbar section (Fig. 8).

Raise the lumbar section by pressing the lever downwards (for the therapist's convenience, the lever is located on both sides under this table section).

Perform the vertebral drop technique in the lumbar region through a quick and vigorous pressure that will force the table section to drop.

Drop for the pelvic section (Fig. 9).

Raise the pelvic section by pressing the lever downwards (for the therapist's convenience, the lever is located on both sides under this table section).

Perform the pelvic drop technique with a quick and vigorous pressure that will force the table section to lower.

Fig. 6

Fig. 7

Fig. 8

Fig. 9

The reaction force of the drop technique can be adjusted by setting the required range with the knob on the right side for the cervical segments and on both sides of the thoracic, lumbar and pelvic segments.

7. Bend adjustment for the pelvic segment and lower limbs

The motion function is carried out with the use of two gas springs.

To move sideways, unlock the adjustment system by moving the yellow lever (Fig. 10) up. To lock the movement in the required position, block the system by moving the lever down.

When moving, use the handle (Fig. 11).

To unlock the vertical movement of the table, use the yellow lever (Fig. 12) by moving it upwards. To lock the movement in the required position, block the system by moving the lever down.

When moving, use the handle (Fig. 11).

Fig. 10 | Fig. 12

Fig. 11

8. Realization of the lumbar traction.

For horizontal movement (traction), unlock the knob on the right side of the frame. After finishing work in this segment, lock the knob according to the diagram.


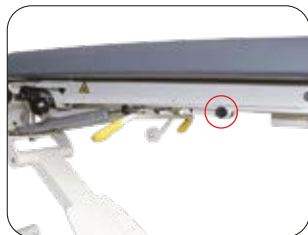
Fig. 13 | Fig. 14

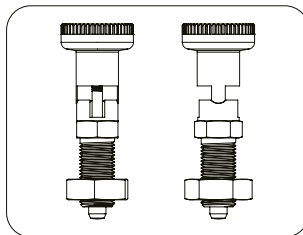9. Footrest adjustment.

The footrest adjustment system allows to obtain a position convenient for the patient through the possibility of two-plane movement.

The movement of the footrest is carried out after unlocking the protection on the left side, additionally pressing the yellow lever unlocks the possibility of movement of the entire footrest frame.

Fig. 15 | Fig. 16

Fig. 17 | Fig. 18

10. Chassis.

The running system is started by lifting the table with the foot lever. After pressing the lever down, the table supported on wheels can be moved. In order to lock the table in the selected position, lower the table on the feet by lifting the lever to the upper position.

11. Safety switch.

The table is equipped with a safety switch (Fig. 19). After turning the key to the "closed padlock" position, the power supply to the table is turned off.

Fig. 19

VII. UTILIZATION

In accordance with the provisions of the law on electrical and electronic equipment waste, it is prohibited to dispose of used equipment marked with the symbol of the crossed out bin with other household waste. Used equipment should be returned to proper collection point. The above statutory obligations were introduced in order to limit the amount of waste generated from electrical and electronic equipment waste and ensure an appropriate level of collection, recovery and recycling of used equipment. The correct implementation of these obligations is important especially when used equipment contains hazardous components that have especially negative impact on the environment and human health.

VIII. MEANING OF PICTOGRAPHS

Attention

Maximum load

Medical device

Authorized Representative
in the EC

Keep dry during storage

CE mark - product compliance with the
requirements of the Medical Devices
Regulation (MDR) 2017/745/EU

Importer

Unique product identification code

Manufacturer's name and address

See the instruction manual for use

Date of manufacture

IX. WARRANTY CONDITIONS

1. For the purposes of the guarantee:
 - a) "Manufacturer" refers to HABYS Spółka z ograniczoną odpowiedzialnością (Limited Liability Company) with its registered seat in Jasło, ul. Produkcyjna 16, 38-200 Jasło, entered into the Register of Entrepreneurs kept by the District Court in Rzeszów, XII Commercial Division of the National Court Register, with the assigned KRS number: 0000513317, with the share capital in the amount of PLN 10.750.000 (ten million seven hundred fifty thousand zlotys), the capital has been fully paid up, NIP: 6852208438, REGON: 180186290.
 - b) "Buyer" refers to the final Buyer of the equipment, manufactured by the Manufacturer, including both the consumer and the entrepreneur.
 - c) "Distributor" refers to an entity that, under an agreement with the Manufacturer, sells the Equipment in a designated area.
 - d) "Authorized Service Centre" refers only to a service held by the Manufacturer or a service indicated by the Manufacturer and held by its Distributor,
 - e) "Equipment" refers to a movable item, manufactured by the Manufacturer, being the subject of the sales and to which the said warranty applies.
2. The manufacturer ensures good quality and efficient operation of the equipment for which the warranty card has been issued, for the period of: 2 years for constructional elements, 2 years for upholstered elements and 2 years for electronic and hydraulic accessories.
3. The guarantee repair does not include the activities provided for in the manual, which the Buyer is obliged to perform on his own (installation of the Equipment, maintenance of the Equipment, etc.).
4. Liability under the guarantee covers only defects resulting from causes inherent in the Equipment. In particular, the guarantee does not cover any damage resulting from improper or inconsistent use, storage, maintenance or transport of the Equipment, as well as colour differences between the parts exposed and not exposed to solar radiation, the influence of chemical or abrasive agents, atmospheric pollution or accidental punctures and abrasions, the presence of stains or rings from greasy or dying substances (fats, creams, lotions or oils), permanent staining with dyes such as inks, permanent markers or other non-permanent dyes used in the production of clothing (e.g. dyes used to make blue jeans).
5. The guarantee does not cover any mechanical damage to the Equipment.
6. The buyer loses the guarantee rights in the event of:
 - a) the Manufacturer states that the Equipment was repaired by entity other than the Authorized Service,
 - b) making any design modifications to the Equipment without the prior written consent of the Manufacturer,
 - c) using the Equipment contrary to its intended use,
 - d) using the Equipment after revealing any defect in it.
7. Regardless of the loss of the guarantee rights, described above, in point 6, the Manufacturer shall not be liable for any damage to property or caused to third parties, which may be caused by the use of the Equipment covered by the guarantee.
8. The guarantee does not exclude or limit the rights resulting from the warranty for defects. Exercising the rights under the guarantee does not affect the seller's liability under the warranty for defects. If the Buyer exercises the rights under the guarantee, the time limit for the exercise of the rights under the warranty for defects is suspended on the date the Manufacturer is notified of the defect. The period runs further from the date of the Manufacturer's refusal to perform the obligations arising from the guarantee or the ineffective expiry of the time for their performance.
9. The scope of the guarantee coverage is limited to the territory of the country in which the Equipment was sold.
10. The Manufacturer shall perform its obligations under the guarantee (removal of defects or damage to the structure of the Equipment arising from reasons inherent in the Equipment, and disclosed during the guarantee period) free of charge within 14 days from the date of delivery of the Equipment to the Manufacturer or the Authorized Service Centre. If the repair of the Equipment cannot be performed within the above-mentioned period, the Manufacturer shall inform the Buyer of the above, and also indicate the reasons for the delay and the expected date of fulfilment of the recognized guarantee request.
11. The use of the guarantee rights requires the total (cumulative) fulfilment of four conditions:
 - a) presenting a VAT invoice, receipt or other proof of purchase of the equipment to the Manufacturer, photographic documentation of the defective Equipment, together with a brief description of the reasons for submitting a guarantee claim,
 - b) reporting the defect within 14 (fourteen) days of its detection,
 - c) delivering the Equipment in original packaging at the expense of the Manufacturer, to the address indicated above in the guarantee card or to the address of the Authorized Service Centre,
 - d) designation of the place where the Manufacturer delivers the equipment after repairing it (limited to the country in which the Equipment was purchased).
12. The guarantee period is extended for the duration of the repair, starting from the date of delivery of the Equipment to the Manufacturer or to the Authorized Service Centre, to the date on which the Equipment was returned to the Buyer.
13. The responsibility of the Manufacturer or the Authorized Service Centre is to define the scope and the method of repairing the Equipment.
14. The guarantee is limited to the repair of the Equipment. The Manufacturer reserves the right to replace the Equipment with a non-defective one, if the repair exceeds the price of the Equipment indicated in the proof of purchase. Under the guarantee, the Buyer shall not be entitled to demand that the Equipment be replaced with a new one. The guarantee, in particular, does not cover liability for losses or damages incurred by the Buyer in connection with the fact that one is not able to use the Equipment.
15. The Manufacturer shall not be liable for any damage to the Equipment caused during the performance of the transport service by the delivery company until it is collected by the Manufacturer from the delivery company.
16. If the shipment of the Equipment was ordered by the Manufacturer, the Buyer is obliged to check the technical condition of the Equipment upon its receipt. If the Buyer finds that the Equipment has been damaged during the transport, the Buyer is obliged to:
 - a) immediately, not later than within 3 days from the date of receipt of the shipment, notify the Manufacturer of any damage to the Equipment that occurred during transport, in order to enable the Manufacturer to submit a complaint to the delivery company and pursue related claims,
 - b) provide the Producer with a complaint report drawn up between the Buyer and the delivery company.
17. The guarantee regulations contained in this document are the sole and exclusive guarantee rules for HABYS limited liability products.

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JIANGSU SINTANG HEALTH & TECHNOLOGY CO., LTD