User Manual and Warranty for Therapeutic Massage Tables

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Estella ERGO 2

Thank you for choosing HABYS products, and congratulations on your successful purchase. Please remember that proper use and regular maintenance of the equipment will ensure full satisfaction and long-lasting performance for many years to come

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PL POLSKI

strona 12

*** Read the instructions carefully before using the product.***

This manual contains detailed guidelines for operation and maintenance. Keep this document for future reference and ensure that all potential users are familiar with its contents before operating the device.

I. INTRODUCTION

The Estella ERGO 2 stationary tables have been designed for professional use in physiotherapy clinics, therapeutic massage, rehabilitation centers, and medical facilities. Thanks to their ergonomic design, wide range of adjustments, and stable base, they provide comfort for both the patient and the therapist. The aesthetic finish and robust construction make them a perfect match for the interiors of modern treatment rooms.

II. INSTALLATION

Unpacking the Product

Before using the table, carefully unpack it, paying special attention to the structural components and upholstered surfaces to avoid accidental damage.

Post-Transport Technical Inspection

After unpacking, carefully inspect the product to ensure it has not been damaged during transport. If any damage is detected, do not connect the table to the power supply — contact the seller immediately to report the issue and initiate a claim.

Verification of Set Completeness

Ensure that all accessories included in the order have been delivered in accordance with the specification. In case of any missing items or discrepancies, please report them to the seller immediately.

Storage and Operating Conditions for Electrically Controlled Tables

Tables equipped with electric control systems should be stored and used exclusively in dry, enclosed spaces with climate control. Recommended environmental conditions:

- Ambient temperature: from +10°C to +40°C
- Relative humidity: from 30% to 75%

Procedure in Case of Exposure to Low Temperatures

If the table has been stored in conditions where the temperature dropped below 0°C, ensure that the room temperature is properly regulated before using the device. It is recommended to maintain room temperature for at least 5 hours prior to connecting the equipment to the power supply.

Guidelines:

- The device is intended for use exclusively by adults with appropriate training.
- Children and inexperienced individuals should not use the treatment table.
- The device should be switched off and disconnected from the power supply when not in use.
- Do not leave the table unattended while it is connected to the mains.
- After completing work, disconnect the power cord.

III. CONNECTING TO THE POWER SUPPLY

Before plugging the device into the electrical outlet, please ensure that:

- The outlet is properly grounded and complies with applicable regulations.
- The outlet can withstand the maximum power load of the device.
- The outlet matches the plug type. If not, the outlet must be replaced.
- The mains voltage corresponds to the specifications provided in the Technical Data table.
- Do not use extension cords.
- Use an unbent power cable.
- The power cord may only be replaced by an authorized specialist.
- Do not attempt to repair the electric actuators yourself.
- Avoid direct sunlight, UV radiation, rain, excessive dust, and moisture.

Warning! The manufacturer disclaims all liability if the above guidelines are not followed.

IV. OPERATION

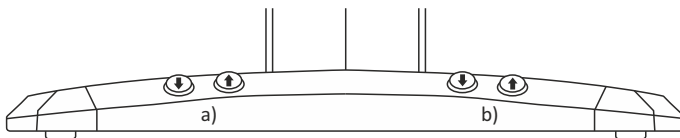
Connect the power cord to an appropriate external power source within the range of 220 V to 240 V. The Estella ERGO 2 table is electrically operated using a hand-held remote control. The adjustment functions include:

1. **Electric backrest adjustment** – Smooth inclination range from 0° to 66°, ensuring a comfortable position for both treatment and relaxation.
2. **Height adjustment** – Enables setting the table height between 62 and 82 cm from the floor, tailored to the therapist's ergonomic needs.
3. **Quick reset function** – Instantly returns the table to two pre-programmed working positions, simplifying operation and saving time.
4. **Tabletop heating system** – Provides pleasant warmth during treatments, enhancing client comfort and supporting therapeutic outcomes.

NOTE!

The perceived temperature of the heating is influenced by the ambient temperature. If the ambient temperature is high, the heating speed is faster and the perceived end temperature is slightly higher than the standard temperature. If the ambient temperature is low, the heating speed is slower and the perceived end temperature is slightly lower than standard.

Control function via buttons integrated in the product frame:

- a) **Electrically adjustable backrest angle:** 0–66°.
- b) **Electrically adjustable seat height:** 62–82 cm from the floor.

Set Contents:

The package also includes:

- A headrest cushion and a round plug for the face opening — these elements can be used interchangeably depending on user preference and the type of treatment performed.
- A Velcro-secured half-roll, which can be conveniently attached under the backrest to provide additional support and comfort during procedures.

V. WARNINGS AND SAFETY PRECAUTIONS

- Use the chair according to the manual.
- Damage caused by misuse is not covered by warranty.
- Not intended for persons with reduced physical or mental capabilities unless supervised.
- Only adults may use the chair.
- Indoor use only, for one person at a time.
- Do not move the chair with a patient seated.
- Do not sit abruptly or jump onto the chair.
- Do not sit, kneel, or lean on the headrest, shelves, or armrests.
- Avoid prolonged contact of upholstery with oils – use protective covers.
- Keep away from fire and heat sources.
- Avoid prolonged sunlight exposure to prevent discoloration.
- After moving from a cold room, wait 15 minutes before connecting to power.
- Fully adjust chair height 2–4 times per week to distribute lubricant inside the motor.
- Do not unplug by pulling the cable.
- Do not disassemble or attempt internal repairs.
- The chair may only be used if all above conditions are met.
- Do not exceed the maximum load specified by the manufacturer.
- Do not use the chair as a bed for sleeping or for any purposes other than massage or cosmetic treatments.

VI. MAINTENANCE AND CARE

*** Disconnect the power cord before cleaning. ***

To keep the surface of the electric beauty chair in perfect condition, it should be cleaned regularly. It is important to carry out cleaning and disinfection before each new user to ensure hygiene and comfort. In addition, a thorough disinfection is recommended once a month to maintain the highest standards of cleanliness and safety.

Metal and plastic surfaces

To keep the metal and plastic surfaces as well as the remote controls in excellent condition, use a soft cloth and a mild cleaning agent. For hard-to-reach areas, such as corners and crevices, use a small brush. While cleaning, wipe the surfaces with the cloth, avoiding excessive amounts of liquid. Then apply a disinfectant according to the manufacturer's instructions and allow the surfaces to dry completely at room temperature.

Seat surface

To keep the chair surface in perfect condition, wipe it with a clean, soft cloth and then apply a special upholstery cleaner evenly. After application, wipe the surface again with a clean, soft cloth to ensure even cleaning. Regular cleaning has a positive effect on the quality and durability of the surface.

Maintenance:

Professional users are responsible for regularly implementing maintenance procedures, both monthly and semi-annual. Control elements and their wiring should be inspected weekly, while the following components must be checked for condition and functionality at least once every six months:

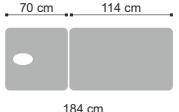
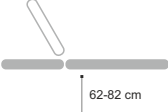
- Power cord and its attachment.
- Motor wiring.
- Proper fastening of accessories.
- Complete all adjustments and ensure that the chair operates correctly.

If any defects are detected, immediately stop using the equipment. For example, if the device makes unusual noises or does not operate correctly, contact the after-sales service. Only experienced or authorized personnel are permitted to open or replace the actuator or the control unit.

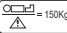
In accordance with the provisions of the law on electrical and electronic equipment waste, it is prohibited to dispose of used equipment marked with the symbol of the crossed out bin with other household waste. Used equipment should be returned to proper collection point. The above statutory obligations were introduced in order to limit the amount of waste generated from electrical and electronic equipment waste and ensure an appropriate level of collection, recovery and recycling of used equipment. The correct implementation of these obligations is important especially when used equipment contains hazardous components that have especially negative impact on the environment and human health.

VII. PRODUCT SPECIFICATIONS

	Dimensions	Possible positions	Weight	Maximum load	Temperature control	Electricity supply
Estella ERGO 2			65 kg	150 kg	Heat level between 35°C and 60°C.	220V-240V

VIII. MEANING OF PICTOGRAPHS

 <p>Attention</p>	 <p>Keep dry during storage</p>	 <p>Manufacturer's name and address</p>
 <p>Maximum load</p>	 <p>CE mark - product compliance with the requirements of the Medical Devices Regulation (MDR) 2017/745/EU</p>	 <p>See the instruction manual for use</p>
 <p>Medical device</p>	 <p>Batch number</p>	 <p>Date of manufacture</p>
 <p>Catalog number</p>	 <p>Unique product identification code</p>	 <p>Made in China</p>

VIII. WARRANTY CONDITIONS.

1. For the purposes of the guarantee:
 - a) "Manufacturer" refers to HABYS Spółka z ograniczoną odpowiedzialnością (Limited Liability Company) with its registered seat in Jasło, ul. Produkcyjna 16, 38-200 Jasło, entered into the Register of Entrepreneurs kept by the District Court in Rzeszów, XII Commercial Division of the National Court Register, with the assigned KRS number: 0000513317, with the share capital in the amount of PLN 10.750.000 (ten million seven hundred fifty thousand zlotys), the capital has been fully paid up, NIP: 6852208438, REGON: 180186290.
 - b) "Buyer" refers to the final Buyer of the equipment, manufactured by the Manufacturer, including both the consumer and the entrepreneur.
 - c) "Distributor" refers to an entity that, under an agreement with the Manufacturer, sells the Equipment in a designated area.
 - d) "Authorized Service Centre" refers only to a service held by the Manufacturer or a service indicated by the Manufacturer and held by its Distributor,
 - e) "Equipment" refers to a movable item, manufactured by the Manufacturer, being the subject of the sales and to which the said warranty applies.
2. **The manufacturer ensures good quality and efficient operation of the equipment, for which the warranty card has been issued, for the period of: 2 years for constructional elements, 2 years for upholstered elements and 2 years for electronic and hydraulic accessories.**
3. The guarantee repair does not include the activities provided for in the manual, which the Buyer is obliged to perform on his own (installation of the Equipment, maintenance of the Equipment, etc.).
4. Liability under the guarantee covers only defects resulting from causes inherent in the Equipment. In particular, the guarantee does not cover any damage resulting from improper or inconsistent use, storage, maintenance or transport of the Equipment, as well as colour differences between the parts exposed and not exposed to solar radiation, the influence of chemical or abrasive agents, atmospheric pollution or accidental punctures and abrasions, the presence of stains or rings from greasy or dyeing substances (fats, creams, lotions or oils), permanent staining with dyes such as inks, permanent markers or other non-permanent dyes used in the production of clothing (e.g. dyes used to make blue jeans).
5. The guarantee does not cover any mechanical damage to the Equipment.
6. The buyer loses the guarantee rights in the event of:
 - a) the Manufacturer states that the Equipment was repaired by entity other than the Authorized Service,
 - b) making any design modifications to the Equipment without the prior written consent of the Manufacturer,
 - c) using the Equipment contrary to its intended use,
 - d) using the Equipment after revealing any defect in it.
7. Regardless of the loss of the guarantee rights, described above, in point 6, the Manufacturer shall not be liable for any damage to property or caused to third parties, which may be caused by the use of the Equipment covered by the guarantee.
8. The guarantee does not exclude or limit the rights resulting from the warranty for defects. Exercising the rights under the guarantee does not affect the seller's liability under the warranty for defects. If the Buyer exercises the rights under the guarantee, the time limit for the exercise of the rights under the warranty for defects is suspended on the date the Manufacturer is notified of the defect. The period runs further from the date of the Manufacturer's refusal to perform the obligations arising from the guarantee or the ineffective expiry of the time for their performance.
9. The scope of the guarantee coverage is limited to the territory of the country in which the Equipment was sold.
10. The Manufacturer shall perform its obligations under the guarantee (removal of defects or damage to the structure of the Equipment arising from reasons inherent in the Equipment, and disclosed during the guarantee period) free of charge within 14 days from the date of delivery of the Equipment to the Manufacturer or the Authorized Service Centre. If the repair of the Equipment cannot be performed within the above-mentioned period, the Manufacturer shall inform the Buyer of the above, and also indicate the reasons for the delay and the expected date of fulfilment of the recognized guarantee request.
11. The use of the guarantee rights requires the total (cumulative) fulfilment of four conditions:
 - a) presenting a VAT invoice, receipt or other proof of purchase of the equipment to the Manufacturer, photographic documentation of the defective Equipment, together with a brief description of the reasons for submitting a guarantee claim,
 - b) reporting the defect within 14 (fourteen) days of its detection,
 - c) delivering the Equipment in original packaging at the expense of the Manufacturer, to the address indicated above in the guarantee card or to the address of the Authorized Service Centre,
 - d) designation of the place where the Manufacturer delivers the equipment after repairing it (limited to the country in which the Equipment was purchased).
12. The guarantee period is extended for the duration of the repair, starting from the date of delivery of the Equipment to the Manufacturer or to the Authorized Service Centre, to the date on which the Equipment was returned to the Buyer.
13. The responsibility of the Manufacturer or the Authorized Service Centre is to define the scope and the method of repairing the Equipment.
14. The guarantee is limited to the repair of the Equipment. The Manufacturer reserves the right to replace the Equipment with a non-defective one, if the repair exceeds the price of the Equipment indicated in the proof of purchase. Under the guarantee, the Buyer shall not be entitled to demand that the Equipment be replaced with a new one. The guarantee, in particular, does not cover liability for losses or damages incurred by the Buyer in connection with the fact that one is not able to use the Equipment.
15. The Manufacturer shall not be liable for any damage to the Equipment caused during the performance of the transport service by the delivery company until it is collected by the Manufacturer from the delivery company.
16. If the shipment of the Equipment was ordered by the Manufacturer, the Buyer is obliged to check the technical condition of the Equipment upon its receipt. If the Buyer finds that the Equipment has been damaged during the transport, the Buyer is obliged to:
 - a) immediately, not later than within 3 days from the date of receipt of the shipment, notify the Manufacturer of any damage to the Equipment that occurred during transport, in order to enable the Manufacturer to submit a complaint to the delivery company and pursue related claims,
 - b) provide the Producer with a complaint report drawn up between the Buyer and the delivery company.
17. The guarantee regulations contained in this document are the sole and exclusive guarantee rules for HABYS limited liability products.

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