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Instruction manual & warranty for stationary massage tables

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Please read the manual and the included safety instructions before using the table.

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Seite 16

I. APPLICATION

The stationary massage tables HABYS Evero enable the physiotherapists to perform correct, safe and effective rehabilitation treatments, physiotherapy, physical therapy and massage aiming at treating or alleviating the course of illnesses and the results of injuries/impairments. The frame of the rehabilitation tables Evero enables an easy and complete access to the patient from each side. The tables are also suitable for use by masseurs performing non-medical massages (sports, beauty, relaxation).

II. INSTALLATION

1. Unfolding.

- Unfold the table.
- Check whether it has not been damaged during transportation. Should the table be damaged, do not install the table and inform the seller immediately.

2. Connection to the power supply.

Before inserting the plug into the power socket, make sure that:

- The socket is properly grounded and complies with the binding regulations;
- The socket may sustain a load of the maximum power output of the device, given in the chart of the Technical Data;
- The voltage of the electrical power is in accordance with the data given in the chart of the Technical Data;
- The plug fits into the socket. Otherwise the socket needs to be replaced.

- Do not install the devices in the open air.
- Electric actuators may work no longer than 2 minutes.
- The device can be restarted after a 18-minute cooldown period.
- After installation of the table the power socket should be easily accessible.
- Do not use the extension cords.
- The cable should not be bent.
- The cable of the power supply may be replaced only by an authorised specialist.
- Independent repairs of the electric actuators cannot be performed.
- Avoid direct sunlight, UV radiations, rain, extensive dust, moisture.


Attention!

The manufacturer declines all responsibility if the user doesn't follow the above mentioned rules.

III. OPERATION

1. Electric height adjustment by using a hand-operated remote control.


The tables are raised and lowered using a hand-operated remote control as described below: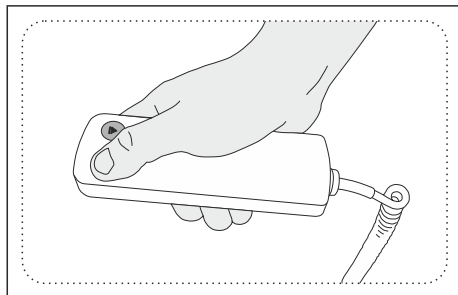

Control is operated by using of pressing and holding the one of the buttons marked with arrows – up/down.

2. Height adjustment by means of walking-free foot controller (option available on Evero Cross X4 (F) and Evero Cross X5 models).

The tables are raised and lowered using a walking-free foot controller as described: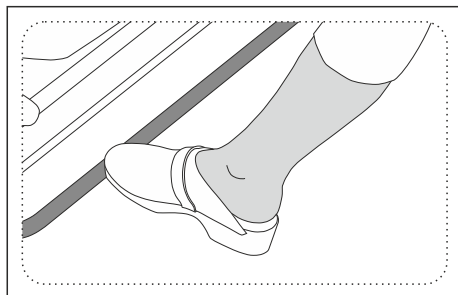


Control takes place by using of moving the walking-free foot control vertically (up/ down) with a foot. The walking-free foot control is located around the base of the table.

By pushing the bar down, the tabletop goes up.
By lifting the bar upwards, the tabletop is going down.

3. Headrest adjustment.


Headrest adjustment takes place by using a gas spring: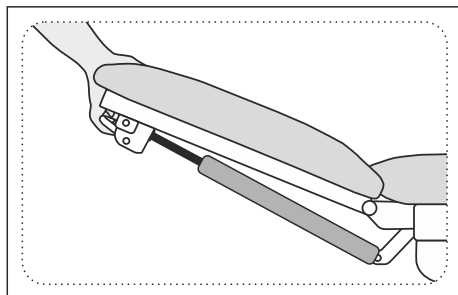

Adjustment range: -55° to $+35^{\circ}$

After releasing the lever, the headrest needs to be placed in the required position.


4. Armrests adjustment.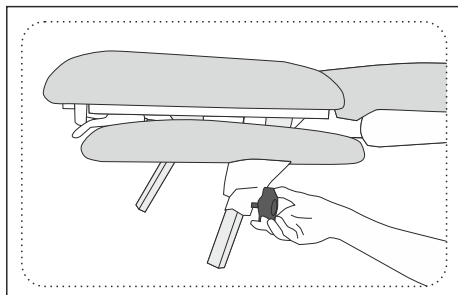

Adjustment range: 0–150 mm

To lower the armrest down, unscrew the knob counterclockwise and then adjust the height to the required position. Then tighten the knob clockwise. Proceed similarly to raise the armrest.


5. Adjustment of table top (option available on Evero Cross X5 model).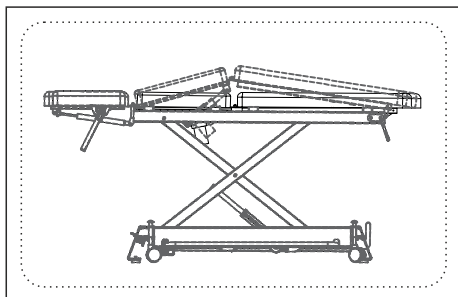

Adjustment of the middle section of the table.

Adjustment range: 0° to 15°

The angle of the bed is adjusted using the button located under the tabletop. To set the desired position, press and hold the button marked with arrows – up/down.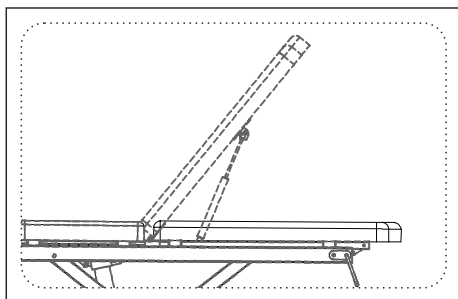


Adjustment of the back section of the table.

Adjustment range: 0° to 60°

The back section is adjusted using a gas spring. Press and hold the lever located at the back of the table to adjust the angle, then release the lever to lock the back section in the desired position.

6. Chassis.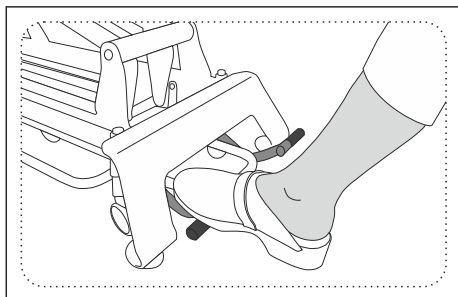

The chassis is activated by raising the table with a foot lever. After pressing the lever down, table supported on wheels can be moved. In order to lock the table in the required position, lower the table so that it is supported on feet, by lifting lever to the upper position.


Sitting on the headrest, armrests or side shelves is forbidden.

EN


7. Safety switch.

Switch is located on the remote control (option available on the Evero Cross X4 (R) model).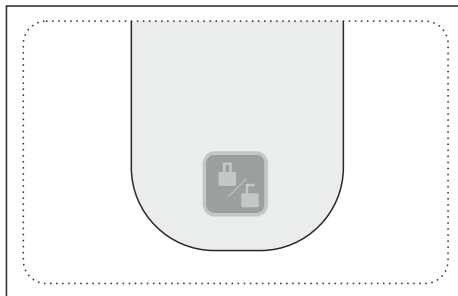

The table is equipped with a safety switch located on the remote control. After pressing and holding for 3 seconds, the table raising and lowering function is blocked. To unlock the function, hold the button down again for another 3 seconds.

Magnetic safety key is located in the table frame (option available on the Evero Cross X4 (F) and Evero Cross X5 models).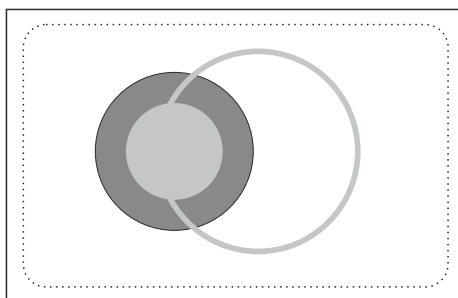

The table is equipped with a magnetic safety key located in the front part of the table frame. The functions for raising/lowering the table and adjusting the angle of the middle section are active only when the key is inserted into the dedicated slot. After removing the key, the functions for raising/lowering the table and adjusting the angle of the middle section remain inactive.

IV. RECOMMENDATIONS AND SAFETY RULES

- The table should be operated in accordance with its instruction manual.
- Damages caused by failure to follow the above recommendations shall not be grounds for a complaint.
- The device is not intended for use by persons with limited physical or mental abilities or those without sufficient experience and knowledge, unless they are under proper supervision or have received instructions from a responsible person.
- The table should only be used by adults in accordance with the instruction manual.
- The table is intended for use in dry, enclosed rooms and for the treatment of only one person at a time.
- The table must not be moved while a patient is on it.
- Rapid sitting on the table (jumping on) is forbidden.
- Sitting, kneeling and leaning on the headrest, shelves and armrests is forbidden.
- Raise the headrest before lowering the table to avoid possible contact with the floor.

- If the patient is laid on the table without clothing, the therapist should use disposable covers or disinfect the table.
- Long-term contact of the table upholstery with oils should be avoided. For that purpose oilresistant covers or water-resistant sheets should be used.
- The table should be used and stored far away from the direct sources of fire and heat.
- The table should not be exposed to long-term solar radiation. It may cause discolouration of the table upholstery.
- The plug should not be removed from the power socket by pulling the cable.
- The device should not be disassembled and there should be no attempts to get into it.
- After moving the table from a cold environment, wait 15 minutes before connecting it to the power supply.
- Only after meeting the above recommendations there are no contraindications to use the product.
- Each serious incident related to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

V. DISINFECTION, MAINTENANCE AND CLEANING

Steel and lacquered elements: Products that do not contain abrasives are recommended for cleaning such elements.

Upholstery:

- Do not use: pastes, waxes, sprays, strong detergents, agents containing solvents, cleaning agents for natural leather.
- Dirt from oils and creams must be removed immediately using a mild detergent solution (warm water with potassium soap) and a soft cloth or sponge.
- For daily cleaning, use a mild detergent, preferably a potassium soap solution, applied with a sponge or soft brush.
- Stronger spot soiling should be cleaned using a 25% solution of ethyl alcohol. Wipe gently with soaked gauze swab. At the end, wipe the cleaned place with a damp cloth and then wipe dry.







Disinfection: Disinfect occasionally and if necessary disinfectants dedicated to PVC, PU upholsteries.

Attention: Before using any cleaning agent other than a mild detergent, test it on an inconspicuous area first.

In accordance with the provisions of the law on electrical and electronic equipment waste, it is prohibited to dispose of used equipment marked with the symbol of the crossed out bin with other household waste. Used equipment should be returned to proper collection point. The above statutory obligations were introduced in order to limit the amount of waste generated from electrical and electronic equipment waste and ensure an appropriate level of collection, recovery and recycling of used equipment. The correct implementation of these obligations is important especially when used equipment contains hazardous components that have especially negative impact on the environment and human health.

VI. DIMENSIONS AND WEIGHT

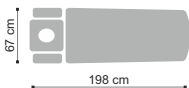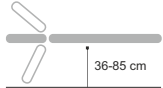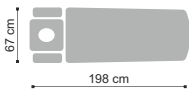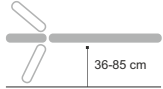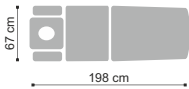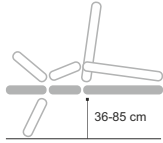	Dimensions	Possible positions	Weight	Maximum load allowed
Evero Cross X4 (R)			73 kg	200 kg
Evero Cross X4 (F)			77 kg	200 kg
Evero Cross X5			93 kg	200 kg

Power supply:

Voltage 230V ~50/60Hz; 150W; 0,7 A.

VII. MEANING OF PICTOGRAPHS

Attention

Keep dry during storage

Manufacturer's name and address

Maximum load

CE mark - product compliance with the requirements of the Medical Devices Regulation (MDR) 2017/745/EU

See the instruction manual for use

Medical device

Unique product identification code

Date of manufacture

Authorized Representative in the EC

VIII. WARRANTY CONDITIONS

1. For the purposes of the guarantee:
 - a) "Manufacturer" refers to HABYS Spółka z ograniczoną odpowiedzialnością (Limited Liability Company) with its registered seat in Jasło, ul. Produkcyjna 16, 38-200 Jasło, entered into the Register of Entrepreneurs kept by the District Court in Rzeszów, XII Commercial Division of the National Court Register, with the assigned KRS number: 0000513317, with the share capital in the amount of PLN 10.750.000 (ten million seven hundred fifty thousand zlotys), the capital has been fully paid up, NIP: 6852208438, REGON: 180186290.
 - b) "Buyer" refers to the final Buyer of the equipment, manufactured by the Manufacturer, including both the consumer and the entrepreneur.
 - c) "Distributor" refers to an entity that, under an agreement with the Manufacturer, sells the Equipment in a designated area.
 - d) "Authorized Service Centre" refers only to a service held by the Manufacturer or a service indicated by the Manufacturer and held by its Distributor,
 - e) "Equipment" refers to a movable item, manufactured by the Manufacturer, being the subject of the sales and to which the said warranty applies.
2. **The manufacturer ensures good quality and efficient operation of the equipment, for which the warranty card has been issued, for the period of: 2 years for constructional elements, 2 years for upholstered elements and 2 years for electronic and hydraulic accessories.**
3. The guarantee repair does not include the activities provided for in the manual, which the Buyer is obliged to perform on his own (installation of the Equipment, maintenance of the Equipment, etc.).
4. Liability under the guarantee covers only defects resulting from causes inherent in the Equipment., In particular, the guarantee does not cover any damage resulting from improper or inconsistent use, storage, maintenance or transport of the Equipment, as well as colour differences between the parts exposed and not exposed to solar radiation, the influence of chemical or abrasive agents, atmospheric pollution or accidental punctures and abrasions, the presence of stains or rings from greasy or dyeing substances (fats, creams, lotions or oils), permanent staining with dyes such as inks, permanent markers or other non-permanent dyes used in the production of clothing (e.g. dyes used to make blue jeans).
5. The guarantee does not cover any mechanical damage to the Equipment.
6. The buyer loses the guarantee rights in the event of:
 - a) the Manufacturer states that the Equipment was repaired by entity other than the Authorized Service,
 - b) making any design modifications to the Equipment without the prior written consent of the Manufacturer,
 - c) using the Equipment contrary to its intended use,
 - d) using the Equipment after revealing any defect in it.
7. Regardless of the loss of the guarantee rights, described above, in point 6, the Manufacturer shall not be liable for any damage to property or caused to third parties, which may be caused by the use of the Equipment covered by the guarantee.
8. The guarantee does not exclude or limit the rights resulting from the warranty for defects. Exercising the rights under the guarantee does not affect the seller's liability under the warranty for defects. If the Buyer exercises the rights under the guarantee, the time limit for the exercise of the rights under the warranty for defects is suspended on the date the Manufacturer is notified of the defect. The period runs further from the date of the Manufacturer's refusal to perform the obligations arising from the guarantee or the ineffective expiry of the time for their performance.
9. The scope of the guarantee coverage is limited to the territory of the country in which the Equipment was sold.
10. The Manufacturer shall perform its obligations under the guarantee (removal of defects or damage to the structure of the Equipment arising from reasons inherent in the Equipment, and disclosed during the guarantee period) free of charge within 14 days from the date of delivery of the Equipment to the Manufacturer or the Authorized Service Centre. If the repair of the Equipment cannot be performed within the above-mentioned period, the Manufacturer shall inform the Buyer of the above, and also indicate the reasons for the delay and the expected date of fulfilment of the recognized guarantee request.
11. The use of the guarantee rights requires the total (cumulative) fulfilment of four conditions:
 - a) presenting a VAT invoice, receipt or other proof of purchase of the equipment to the Manufacturer, photographic documentation of the defective Equipment, together with a brief description of the reasons for submitting a guarantee claim,
 - b) reporting the defect within 14 (fourteen) days of its detection,
 - c) delivering the Equipment in original packaging at the expense of the Manufacturer, to the address indicated above in the guarantee card or to the address of the Authorized Service Centre,
 - d) designation of the place where the Manufacturer delivers the equipment after repairing it (limited to the country in which the Equipment was purchased).
12. The guarantee period is extended for the duration of the repair, starting from the date of delivery of the Equipment to the Manufacturer or to the Authorized Service Centre, to the date on which the Equipment was returned to the Buyer.
13. The responsibility of the Manufacturer or the Authorized Service Centre is to define the scope and the method of repairing the Equipment.
14. The guarantee is limited to the repair of the Equipment. The Manufacturer reserves the right to replace the Equipment with a non-defective one, if the repair exceeds the price of the Equipment indicated in the proof of purchase. Under the guarantee, the Buyer shall not be entitled to demand that the Equipment be replaced with a new one. The guarantee, in particular, does not cover liability for losses or damages incurred by the Buyer in connection with the fact that one is not able to use the Equipment.
15. The Manufacturer shall not be liable for any damage to the Equipment caused during the performance of the transport service by the delivery company until it is collected by the Manufacturer from the delivery company.
16. If the shipment of the Equipment was ordered by the Manufacturer, the Buyer is obliged to check the technical condition of the Equipment upon its receipt. If the Buyer finds that the Equipment has been damaged during the transport, the Buyer is obliged to:
 - a) immediately, not later than within 3 days from the date of receipt of the shipment, notify the Manufacturer of any damage to the Equipment that occurred during transport, in order to enable the Manufacturer to submit a complaint to the delivery company and pursue related claims,
 - b) provide the Producer with a complaint report drawn up between the Buyer and the delivery company.
17. The guarantee regulations contained in this document are the sole and exclusive guarantee rules for HABYS limited liability products.

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DONGPIN BEAUTY AND MEDICAL CO.,LTD.