**Instruction manual & warranty
for oil and massage stones heater**

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Thank you for choosing HABYS products and congratulations on your purchase. We recall that the proper use and maintenance of the products ensure full satisfaction of their use for many years.

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

strona 12

I. APPLICATION.

The device is designed for heating basalt stones and massage oils. It is a modern heater equipped with an electronic timer and certified with the CE mark.

EN

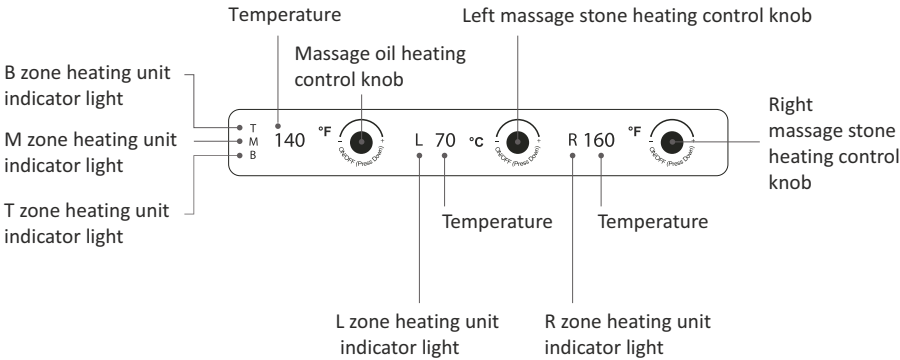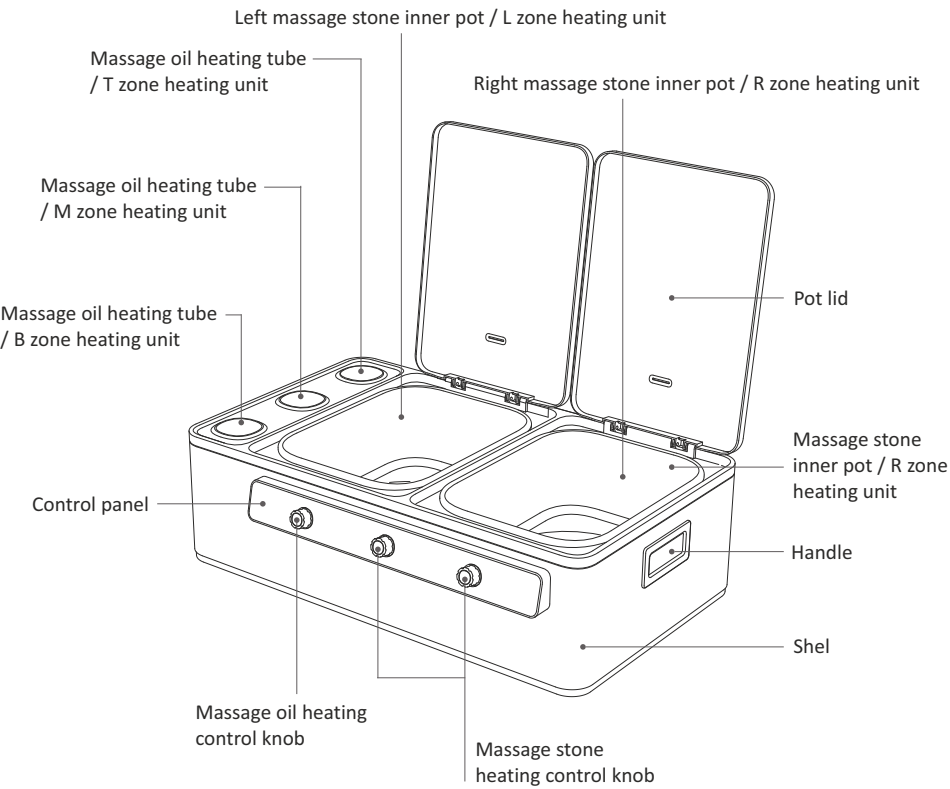II. PRODUCT STRUCTURE.

III. PREPARATION.

1. Place the device on a horizontal surface and connect the power plug to an external socket;
2. Place a towel on the bottom of the inner pot, then gently put the massage stone in, add water to soak the stone, the water level should not exceed the maximum capacity line, and cover the pot;
3. Place the oil bottle containing massage oil or lotion into the massage oil heating tube T, M or B.

Power On:

1. [Short press] the right massage stone heating control knob to turn on the right massage stone inner pot;
2. [Short press] the left massage stone heating control knob to turn on the left massage stone inner pot;
3. [Short press] the massage oil heating control knob to turn on the massage oil heating tube.

Temperature Adjustment:

1. Rotate the right massage stone heating control knob to adjust the set temperature of the right massage stone inner pot;
2. Rotate the left massage stone heating control knob to adjust the set temperature of the left massage stone inner pot;
3. Rotate the massage oil heating control knob to adjust the set temperature of the massage oil heating tube;
4. Turn clockwise to increase the temperature set value, and turn counterclockwise to decrease the temperature set value. The screen to the left of the adjustment knob on the control panel will display the set temperature value.

Shut Down:

1. When the right massage stone inner pot is heating, short press the right massage stone heating control knob and the heating of the right massage stone inner pot will be turned off;
2. When the left massage stone inner pot is heating, short press the left massage stone heating control knob and the heating of the left massage stone inner pot will be turned off;
3. When the massage oil heating tube is heating, short press the massage oil heating control knob, and the heating function of the massage oil heating tube will be turned off.

Switch Temperature (Celsius°C/Fahrenheit°F):

1. Press and hold for 3 seconds on any one knob of the three to switch the temperature type displayed.

Temperature Test Before Use:

1. Before taking the massage stone out of the container, please use a thermometer to measure the temperature and make sure the temperature of the massage stone is below 65°C/150°F;
2. Please use a spoon or colander to remove the massage stone from the inner pot of the massage stone. Do not take out the massage stone directly from the inner pot with your hands;
3. Before using a massage stone or massage oil for massage, touch it with your hands to feel the temperature. Make sure the temperature is safe for the human body before using it on customers to avoid scalding due to excessive temperature.

Temperature Display on the Control Panel:

1. After setting the preset temperature of the massage stone heating zone, the screen on the left side of the massage stone heating control knob will display the current temperature of the unit after 5 seconds; (L zone represents the massage stone heating pot on the left side; R zone represents the massage stone heating pot on the right side).
2. After setting the preset temperature of the massage oil heating zone, the screen on the left side of the massage oil heating control knob will display the temperatures of the three massage oil heating tubes after 5 seconds; (T zone represents the massage oil heating tube farthest from the user/M zone represents the massage oil heating tube in the middle; B zone represents the massage oil heating tube closest to the user);

3. Unit indicator light (T/M/B/L/R): red light indicates heating; green indicates the set temperature has been reached; yellow indicates the set temperature is exceeded by 5°C (or 10°F).

Temperature adjustment range:

- Massage oils: 40–60 °C / 104–140 °F
- Massage stones: 40–70 °C / 105–160 °F

IV. SAFETY RULES.

1. Do not touch hot surfaces.
2. Do not immerse the cord, plug, or the entire device in water or other liquids.
3. Turn off the device before unplugging it from the power outlet.
4. Do not use the device while bathing.
5. Use extreme caution if children are near the operating device.
6. Unplug the device when not in use or before cleaning.
7. Allow the device to cool down before removing any parts or cleaning it.
8. Do not use the device if the cord or plug is damaged.
9. Modifying the device may lead to fire or electric shock.
10. This device is intended for indoor use only.
11. The power cord should not hang over the edge of a table or come into contact with hot surfaces.
12. Do not place the device on gas burners, electric heaters, or inside ovens.
13. Exercise caution when moving the heater filled with hot water.
14. Use the device only for the purposes described in this manual.
15. The water tank must not be filled beyond 2.5 cm below the top edge.
16. Always place the stones in the container before adding water.
17. Do not fill the device with water while it is connected to a power source.
18. Check the stone's temperature using a thermometer, your fingers, and then hold it in your hand before placing it on a client.
19. To avoid burns, use a special glove or spoon to remove stones from the heater.
20. Do not fill the device with boiling or steaming water to prevent burns.
21. Turn off the power when the device is not in use.
22. This device is intended for professional use only – users must read the manual thoroughly beforehand.
23. If the massage stone's temperature exceeds 55 °C / 130 °F, wrap it in a towel before lifting or using it. Do not touch the stone directly to the skin to avoid burns.
24. The device is designed solely for heating massage oils and stones. The appropriate massage temperature must be determined and is the sole responsibility of the user.
25. The stone heating pot must be filled with water before switching on. Do not operate the heater dry.
26. Use care when lifting and placing stones – do not drop them to avoid damaging the device.
27. Do not wash the device. If it becomes wet, dry it thoroughly before reuse.
28. Wipe the device with a soft cloth or clean the surface with a neutral detergent, then let it air dry.
29. Do not use the device continuously for long periods to prevent electrical failure.
30. Do not immerse the device in water or place it in areas with high humidity or temperature, such as a balcony or bathroom.
31. Do not operate the device with wet hands.
32. The power socket used must support a current of at least 13A.
33. Do not disassemble the device.
34. Store the device out of reach of children to prevent accidents.
35. Only use massage oil bottles made of PP (polypropylene). Bottles made of PET or other materials with lower melting points may melt.
36. The device will automatically shut off after 8 hours of continuous heating or standby. To continue use, turn it off for 1 hour, then restart.

V. SIMPLE FAULT ANALYSIS AND SOLUTION.

Display	Problem	Solution
124	The system is normal.	Continue to use.
Display --	- No massage oil heating bottle is placed in the massage oil heating tube; - The massage stone heating pot is not installed; - Insufficient weight of massage stones and water were placed in the massage stone heating pot.	Place the massage oil bottle into the massage oil heating tube; Place the massage stone heating pot and add enough massage stone and water into it.
T/M/B massage oil unit indicator light is not on.	The massage oil heating bottle is not placed in the massage oil heating tube.	Place the massage oil bottle into the massage oil heating tube.
L/R massage stone unit indicator light does not light up.	- The massage stone heating pot is not installed; - Insufficient weight of massage stones and water were placed in the massage stone heating pot.	Place the massage stone heating pot and add enough massage stone and water into it.
L1, L2, L3, L4, L5	Zone B, M, T, L, R – The temperature of the massage oil heating unit is abnormally low	Please increase the set temperature accordingly.

VI. MAINTENANCE AND CLEANING.

1. Clean the device after each session.
2. Before cleaning, set the temperature control to minimum and unplug the device.
3. Allow the device to cool to room temperature.
4. Carefully remove the stones and then pour out the water.
5. Clean the tank and lid with water and a small amount of soap, then rinse with clean water and dry with a towel.
6. Wipe the heater with a soft cloth. Never immerse the heater in water.
7. Clean the stones with warm water and a small amount of soap or other mild detergent. Rinse the stones with warm water and dry them thoroughly.

In accordance with the provisions of the law on electrical and electronic equipment waste, it is prohibited to dispose of used equipment marked with the symbol of the crossed out bin with other household waste. Used equipment should be returned to proper collection point. The above statutory obligations were introduced in order to limit the amount of waste generated from electrical and electronic equipment waste and ensure an appropriate level of collection, recovery and recycling of used equipment. The correct implementation of these obligations is important especially when used equipment contains hazardous components that have especially negative impact on the environment and human health.

VIII. WARRANTY CONDITIONS.

EN

1. For the purposes of the guarantee:
 - a) "Manufacturer" refers to HABYS Spółka z ograniczoną odpowiedzialnością (Limited Liability Company) with its registered seat in Jasło, ul. Produkcyjna 16, 38-200 Jasło, entered into the Register of Entrepreneurs kept by the District Court in Rzeszów, XII Commercial Division of the National Court Register, with the assigned KRS number: 0000513317, with the share capital in the amount of PLN 10.750.000 (ten million seven hundred fifty thousand zlotys), the capital has been fully paid up, NIP: 6852208438, REGON: 180186290.
 - b) "Buyer" refers to the final Buyer of the equipment, manufactured by the Manufacturer, including both the consumer and the entrepreneur.
 - c) "Distributor" refers to an entity that, under an agreement with the Manufacturer, sells the Equipment in a designated area.
 - d) "Authorized Service Centre" refers only to a service held by the Manufacturer or a service indicated by the Manufacturer and held by its Distributor,
 - e) "Equipment" refers to a movable item, manufactured by the Manufacturer, being the subject of the sales and to which the said warranty applies.
2. **The manufacturer ensures good quality and efficient operation of the equipment for which the warranty card has been issued, for the period of 2 years.**
3. The guarantee repair does not include the activities provided for in the manual, which the Buyer is obliged to perform on his own (installation of the Equipment, maintenance of the Equipment, etc.).
4. Liability under the guarantee covers only defects resulting from causes inherent in the Equipment., In particular, the guarantee does not cover any damage resulting from improper or inconsistent use, storage, maintenance or transport of the Equipment, as well as colour differences between the parts exposed and not exposed to solar radiation, the influence of chemical or abrasive agents, atmospheric pollution or accidental punctures and abrasions, the presence of stains or rings from greasy or dyeing substances (fats, creams, lotions or oils), permanent staining with dyes such as inks, permanent markers or other non-permanent dyes used in the production of clothing (e.g. dyes used to make blue jeans).
5. The guarantee does not cover any mechanical damage to the Equipment.
6. The buyer loses the guarantee rights in the event of:
 - a) the Manufacturer states that the Equipment was repaired by entity other than the Authorized Service,
 - b) making any design modifications to the Equipment without the prior written consent of the Manufacturer,
 - c) using the Equipment contrary to its intended use,
 - d) using the Equipment after revealing any defect in it.
7. Regardless of the loss of the guarantee rights, described above, in point 6, the Manufacturer shall not be liable for any damage to property or caused to third parties, which may be caused by the use of the Equipment covered by the guarantee.
8. The guarantee does not exclude or limit the rights resulting from the warranty for defects. Exercising the rights under the guarantee does not affect the seller's liability under the warranty for defects. If the Buyer exercises the rights under the guarantee, the time limit for the exercise of the rights under the warranty for defects is suspended on the date the Manufacturer is notified of the defect. The period runs further from the date of the Manufacturer's refusal to perform the obligations arising from the guarantee or the ineffective expiry of the time for their performance.
9. The scope of the guarantee coverage is limited to the territory of the country in which the Equipment was sold.
10. The Manufacturer shall perform its obligations under the guarantee (removal of defects or damage to the structure of the Equipment arising from reasons inherent in the Equipment, and disclosed during the guarantee period) free of charge within 14 days from the date of delivery of the Equipment to the Manufacturer or the Authorized Service Centre. If the repair of the Equipment cannot be performed within the above-mentioned period, the Manufacturer shall inform the Buyer of the above, and also indicate the reasons for the delay and the expected date of fulfilment of the recognized guarantee request.
11. The use of the guarantee rights requires the total (cumulative) fulfilment of four conditions:
 - a) presenting a VAT invoice, receipt or other proof of purchase of the equipment to the Manufacturer, photographic documentation of the defective Equipment, together with a brief description of the reasons for submitting a guarantee claim,
 - b) reporting the defect within 14 (fourteen) days of its detection,
 - c) delivering the Equipment in original packaging at the expense of the Manufacturer, to the address indicated above in the guarantee card or to the address of the Authorized Service Centre,
 - d) designation of the place where the Manufacturer delivers the equipment after repairing it (limited to the country in which the Equipment was purchased).
12. The guarantee period is extended for the duration of the repair, starting from the date of delivery of the Equipment to the Manufacturer or to the Authorized Service Centre, to the date on which the Equipment was returned to the Buyer.
13. The responsibility of the Manufacturer or the Authorized Service Centre is to define the scope and the method of repairing the Equipment.
14. The guarantee is limited to the repair of the Equipment. The Manufacturer reserves the right to replace the Equipment with a non-defective one, if the repair exceeds the price of the Equipment indicated in the proof of purchase. Under the guarantee, the Buyer shall not be entitled to demand that the Equipment be replaced with a new one. The guarantee, in particular, does not cover liability for losses or damages incurred by the Buyer in connection with the fact that one is not able to use the Equipment.
15. The Manufacturer shall not be liable for any damage to the Equipment caused during the performance of the transport service by the delivery company until it is collected by the Manufacturer from the delivery company.
16. If the shipment of the Equipment was ordered by the Manufacturer, the Buyer is obliged to check the technical condition of the Equipment upon its receipt. If the Buyer finds that the Equipment has been damaged during the transport, the Buyer is obliged to:
 - a) immediately, not later than within 3 days from the date of receipt of the shipment, notify the Manufacturer of any damage to the Equipment that occurred during transport, in order to enable the Manufacturer to submit a complaint to the delivery company and pursue related claims,
 - b) provide the Producer with a complaint report drawn up between the Buyer and the delivery company.
17. The guarantee regulations contained in this document are the sole and exclusive guarantee rules for HABYS limited liability products.

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